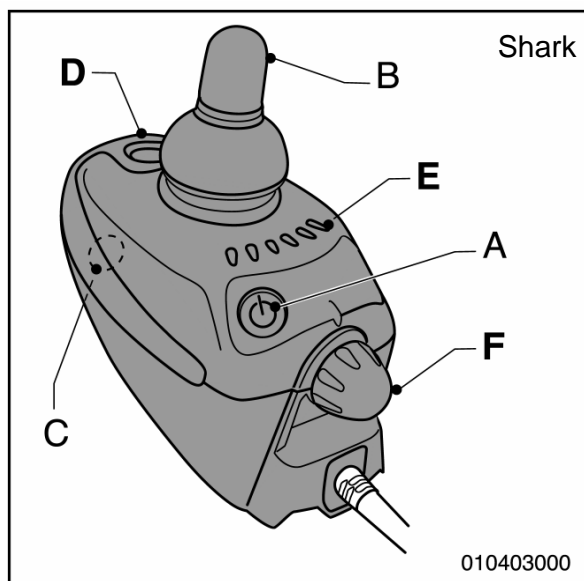
QUICKSTART

IBIS XP

Nederlands	4
English.....	6
Français	8
Deutsch	10
Norsk	12

Bedieningskast

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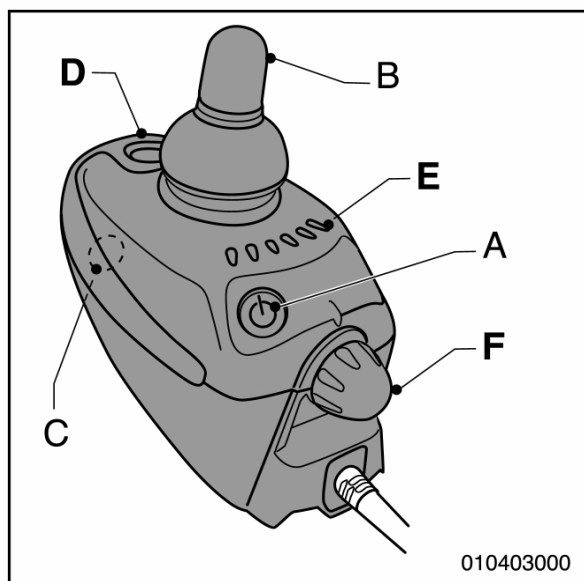
Controller

The controller contains the following components:

- A On/off key
- B Joystick
- C Charge connection
- D Horn
- E Battery indicator
- F Speed control
- G Indicators
- H Lighting
- I Warning lights
- J Actuators/adjustment options

Performance adjustments should only be made by professionals conversant with the adjustment process and the operator's capabilities. Incorrect setting could cause injury to the operator or bystanders, or damage to the vehicle or surrounding property.

The joystick is for driving the wheelchair and adjusting the various electrical options, if installed.

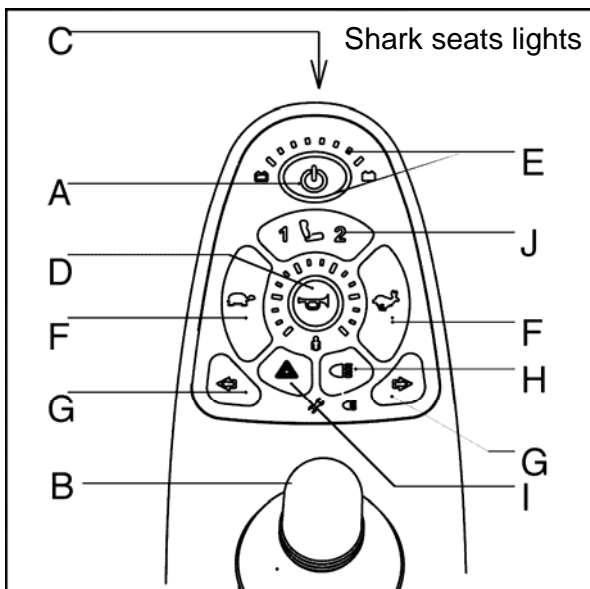
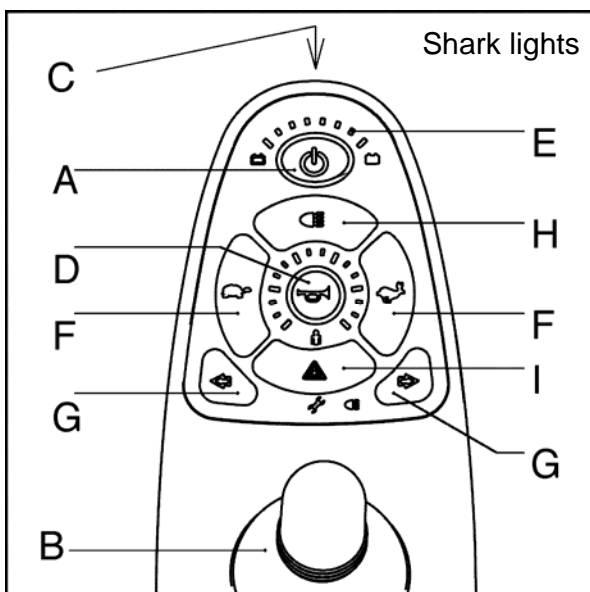
Battery indicator

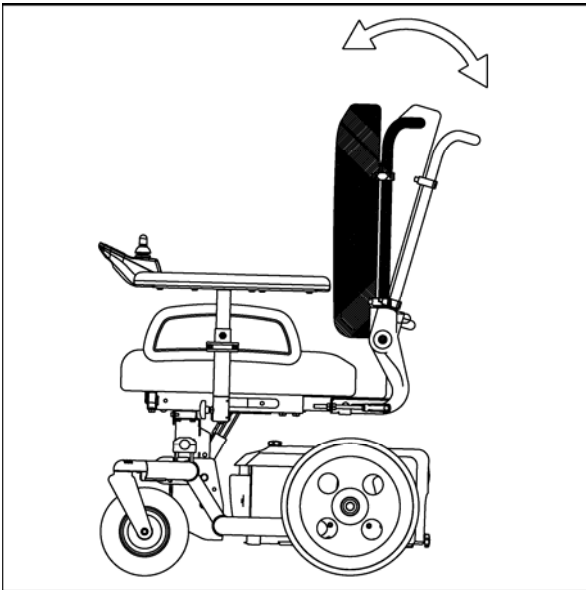
The battery indicator shows the condition of the batteries.

When all the lights are on, the batteries are fully charged. As the batteries are depleted, lights from right to left will go out.

If only the red lights are on, the batteries must be charged immediately.

Electrical adjustment options

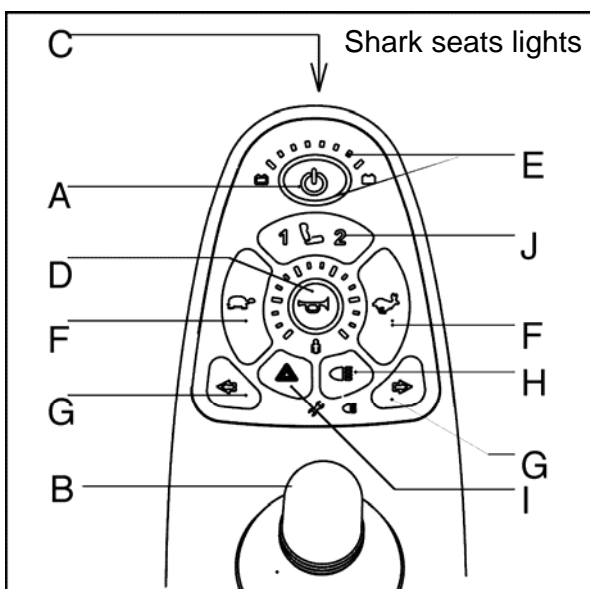
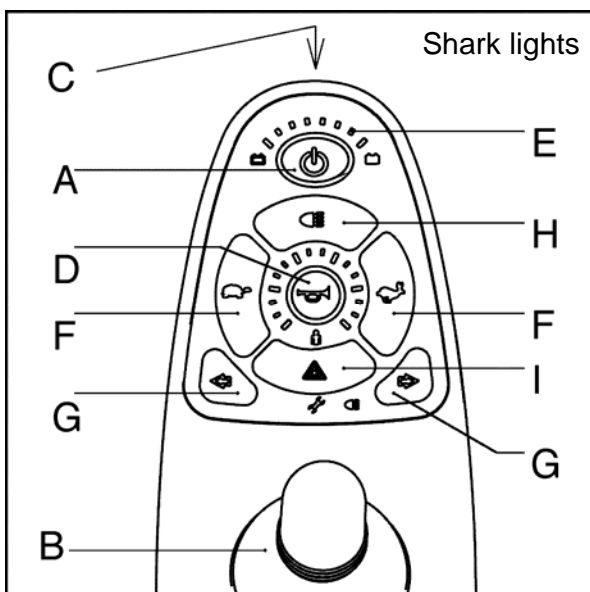
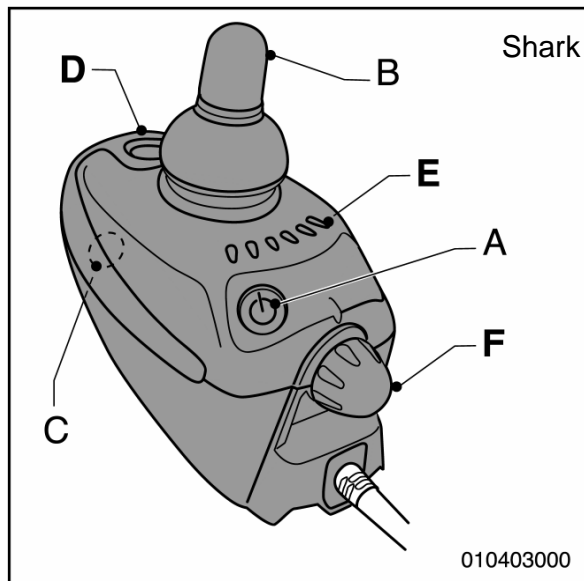
The wheelchair can be equipped with a maximum of 2 electrical adjustment options:

- Electrical backrest recline
- Electrical central legrest
- Electrical high/low
- Electrical seat tilt

If you want to choose between various adjustment options, you have to push the adjustment key (J) several times.

Adjusting:

1. Switch on the wheelchair.
2. Push the adjustment key (J). The indicator light of the key will go on. You can no longer drive the wheelchair.
3. Move the joystick forward: the adjustment option moves forward, or move the joystick backward: the adjustment option moves backward.
4. Push the adjustment key (J) to switch off the adjustment function.

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Handicare B.V.